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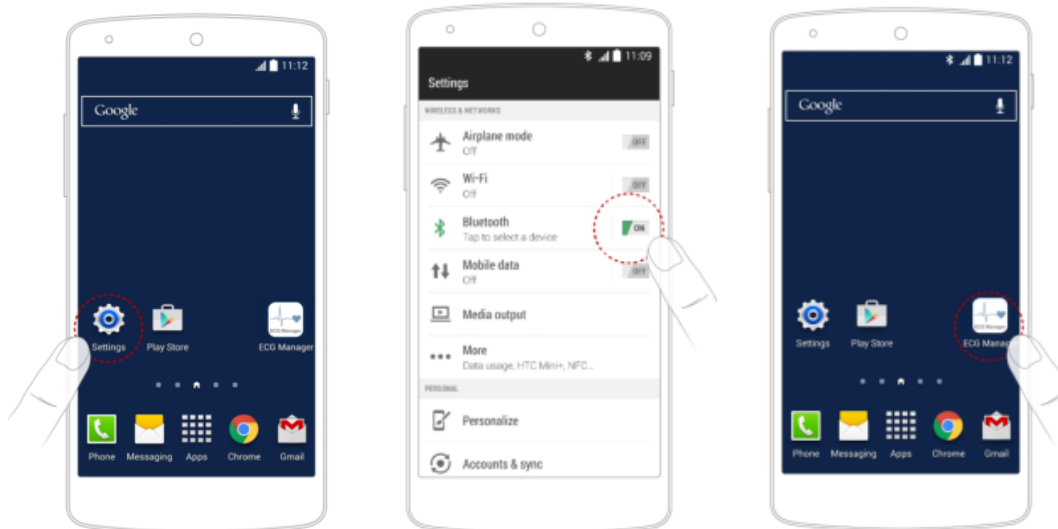
Versionsinformation.


## USING THE *SANITAS* ECG Manager APP

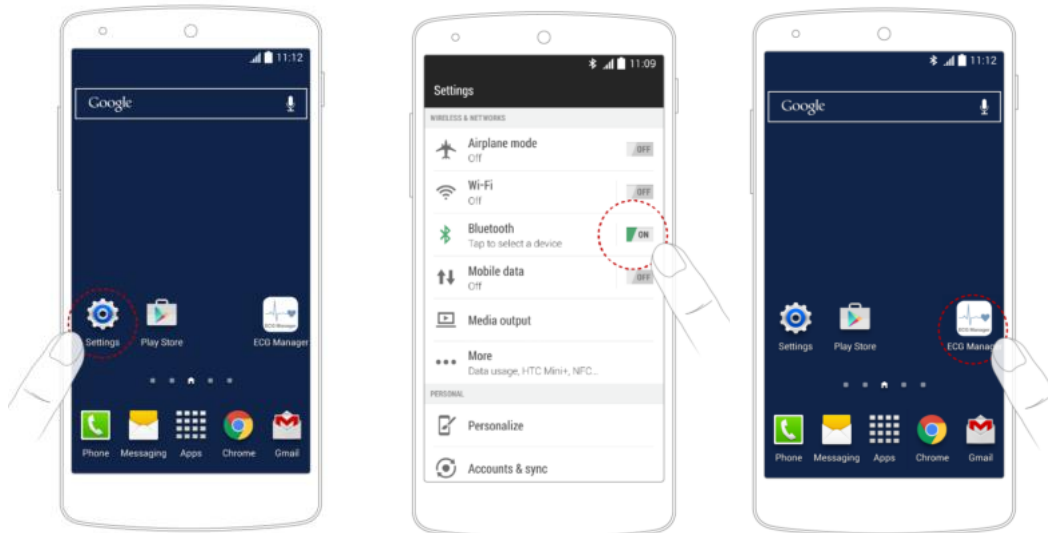It is also possible to transfer the measured values saved on the device to your smartphone using Bluetooth®. You will need the “ECG Manager” app for this. The app is available from the Apple AppStore (iOS) or Google Play (Android).

The data is transferred automatically after the end of recording. The app must be active to allow data transfers. To transfer the values at a later point, go to the „Download“ area of the app.

You can find more information about the transfer of values to the PC software, for example, at [www.sanitas-online.de](http://www.sanitas-online.de) in the FAQ section.

After successfully downloading and installing the app, the icon will appear on the desktop of your smartphone / tablet. Proceed as follows:


1. Open Settings.

2. Turn on Bluetooth in Settings.

3. Open ECG Manager.

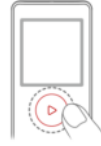
• SME85

4. Turn on Auto-scan.

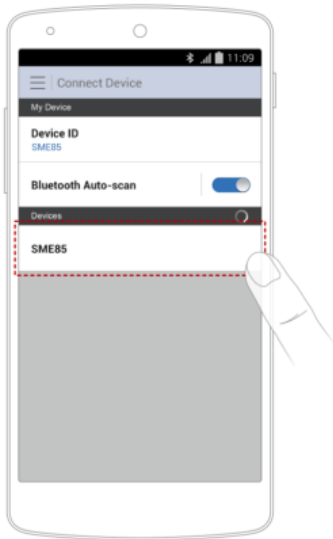5. Turn on your device.

• SME85



6. Tap device id to establish connection.

7. Bluetooth icon will light up.

8. Start your measurement.

## APPLICATION INTERFACE

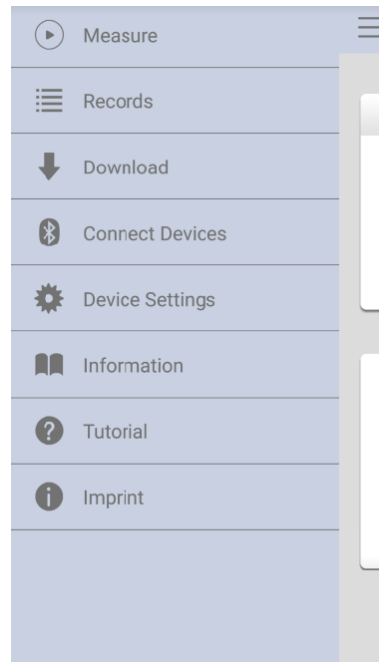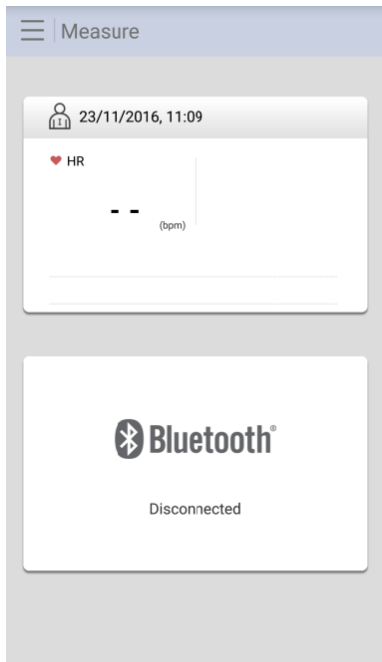A.  Measure

View real time display of your ECG measurement.

B.  Records

View all the ECG records in reverse chronicle order.

Note: If the measurement is taken while Bluetooth is in connection, the records will be stored automatically in your Smartphone/Pad.

C.  Download

Download records from your SME85 to your Smartphone/Pad.

Note: Your SME85 device will be halted during downloading and automatically shut down after download is completed.

E.  Device Settings

Turn on/off the Bluetooth Auto-scan function for connecting the SME85 with your Smartphone/Pad.

F.  Information

Version information.