sanitas

SFT 53

(DE)	Ohrthermometer	
	<i>Gebrauchsanweisung</i>	2
(EN)	Ear thermometer	
	<i>Instructions for use</i>	16
(FR)	Thermomètre auriculaire	
	<i>Mode d'emploi</i>	29
(IT)	Termometro per orecchio	
	<i>Istruzioni per l'uso</i>	43
(RU)	Термометр для измерения температу- ры в ухе	
	<i>Инструкция по применению</i>	56
(PL)	Termometr douszny	
	<i>Instrukcja obsługi</i>	70
(NL)	Oorthermometer	
	<i>Gebruiksaanwijzing</i>	84

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Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Table of contents

1. Signs and symbols	17
2. Intended use	17
3. Notes	18
4. Unit description	19
5. Initial use	20
6. Settings	20
6.1 Setting the time/date	20
6.2 Setting the temperature unit (°C or °F)	21
7. Measuring	21
7.1 Measuring body temperature	21
7.2 Measuring object temperature	23
7.3 Displaying room temperature	24
7.4 Displaying stored measurements	24
8. Replacing the battery	25
9. Cleaning and storage	25
10. Disposal	25
11. What if there are error messages?	26
12. Technical data	27
13. Warranty / service	27

Please read through these Instructions for Use carefully and retain them for future reference; make them available to other users and observe the information they contain.

Included in delivery

Check that the delivery has not been tampered with and make sure that all components are present. Before use, ensure that there is no visible damage to the device or accessories and that all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Services address.

- 1 x ear thermometer
- 2 x 1.5V AAA batteries
- 10 x protective caps
- 1 x set of instructions for use

Subsequent purchase articles

Protective cap set replacement available: 2 x 10 protective caps, item no. 795.26

1. Signs and symbols

The following symbols appear in these instructions for use and on the device:

	WARNING Warning notice indicating a risk of injury or damage to health.
	IMPORTANT Safety note indicating possible damage to the unit/accessory.
	Note Note on important information
	Observe the instructions for use
	Application part, type BF
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
	The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products.
	Permissible storage and transport temperature and humidity

	Permissible operating temperature and humidity
	Manufacturer
IP 22	IP 22, protection against harmful ingress of water and dust
	Dispose of packaging in an environmentally friendly manner.
SN	Serial number

2. Intended use

The ear thermometer is intended for measuring the body temperature in the ear. Do not use the ear thermometer for measurements in other parts of the body.

You can also use the ear thermometer to measure room and object temperatures.

The ear thermometer has been carefully developed for precise, safe and rapid temperature measurements in the ear. It is a non-invasive ear thermometer with an infra-red detector to determine the body temperature in the auditory canal of adults and children.

Use the device for the intended purpose only and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or careless use.

This device complies with the EU Directive 93/42/EEC concerning medical products, the Medizinproduktegesetz (German Medical Devices Act), the ASTM (American Society for Testing and Materials) E 1965-98 and the European Standard EN 12470-5: Clinical thermometers – Part 5: Requirements for infrared ear thermometers (with maximum device) and the European Standard EN 60601-1-2 (in accordance with CISPR 11, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-8), and is subject to particular precautions with regard to electromagnetic compatibility.

Please note that portable and mobile HF communication systems may interfere with this device.

3. Notes

Safety notes

WARNING

- To measure temperature, insert the sensor tip of the ear thermometer carefully into the ear.
- Use of the ear thermometer on different persons can be inappropriate in the event of certain acute infectious diseases because of the possible spread of germs despite cleaning and disinfection. If you have any doubts, please consult your doctor.
- The body temperature measured in the ear may differ from other measurements taken in the mouth, rectum or armpits. Any comparison of these values is therefore futile. Measure temperature regularly to calculate the normal ear temperature and then use these measurements as a basis for

comparison when taking measurements in the event of a suspected high temperature.

- This ear thermometer is intended for measuring the temperature in the ear. Do not use this ear thermometer for measurements in other parts of the body.
- Replace the protective cap after each use to ensure an accurate measurement and prevent cross-contamination.
- The ear thermometer may only be used with protective caps from Sanitas; other protective caps may result in faulty measurements. If you have run out of protective caps, contact the manufacturer or retailer for replacements.
- Consult a doctor if you believe your medical condition to be poor.

Notes on handling batteries

WARNING

- If your skin or eyes come into contact with battery fluid, rinse the affected areas with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Store the batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Never throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.

- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, split or crush the batteries.

Risks to the device

IMPORTANT


- Do not subject the device to mechanical impacts.
- Do not expose the device to direct sunlight.
- Do not expose the device to liquids. The device is not waterproof. Avoid all direct contact with water or other liquids.
- Have the device repaired by authorised service centres only, otherwise its warranty is invalidated.
- Portable and mobile HF communication systems may interfere with this device. More details can be requested from the stated Customer Services address or found at the end of the instructions for use.

Notes on electromagnetic compatibility


- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

4. Unit description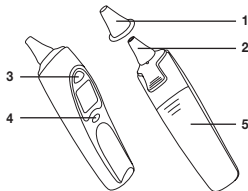

1	Protective cap
2	Measuring sensor
3	START button
4	ON/OFF button 
5	Battery compartment


LCD display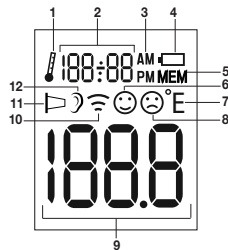

1	Room temperature mode
2	Displays room temperature/ time/date
3	AM/PM display (time)
4	Battery symbol
5	Memory mode
6	No fever symbol
7	Temperature unit (°C or °F)
8	Fever symbol
9	Temperature display
10	Object temperature mode
11	Protective cap symbol
12	Body temperature mode

5. Initial use

The batteries are already inserted in new devices. Before first use, remove the protruding plastic insulating strip from the battery compartment. The ear thermometer will then switch on automatically.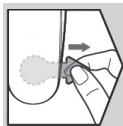

6. Settings

6.1 Setting the time/date

Before beginning to take measurements, we recommend that you set the current time and date. This will allow you to organise the saved measurements more easily later.

1. Press and hold the ON/OFF button  on the switched-off device for three seconds. "24 H" appears on the display.

24 H

2. Use the START button to select the hour format (12h or 24h) and confirm using the ON/OFF button .

12 H




24 H

3. The hour setting appears on the display. Use the START button to select the current hour and confirm using the ON/OFF button .

16:00


4. The minute setting appears on the display. Use the START button to select the current minute and confirm using the ON/OFF button .

16:00

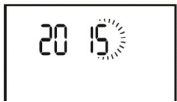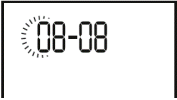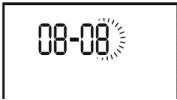
5. The year setting appears on the display. Use the START button to select the current year and confirm using the ON/OFF button  .	
6. The day flashes on the display. Use the START button to select the current day and confirm using the ON/OFF button  .	
7. The month flashes on the display. Use the START button to select the current month and confirm using the ON/OFF button  . The device then switches itself off automatically. The date and time are now saved.	

6.2 Setting the temperature unit (°C or °F)

You can set the device to display the temperature in degrees Celsius (°C) or degrees Fahrenheit (°F).

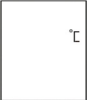1. Press and hold the START button on the switched-off device for three seconds. “°C” appears on the display.	
---	---

2. Use the START button to select the temperature unit (°C or °F) and confirm using the ON/OFF button .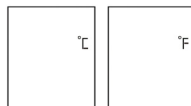

7. Measuring

Note

Check before each use that the lens is intact. If it is damaged, contact your retailer or the service address. Bear in mind that the ear thermometer needs to have been in the room in which the measurement is taken for at least 30 minutes before use.

7.1 Measuring body temperature


WARNING

- Some people produce different readings in their left and right ear. In order to record temperature changes, always measure a person's temperature in the same ear.
- The ear thermometer may be used by children only under adult supervision. Measurement is usually possible over the age of 6 months. In infants under 6 months, the ear canal is still very narrow so the temperature of the eardrum often cannot be recorded and the result displayed is often too low.
- The measurement must not be taken in an ear affected by inflammatory diseases (e.g. discharging pus or secretion), after possible ear injuries (e.g. eardrum damage) or in the


healing period after operative procedures. In all of these cases, please consult your doctor.

- Use of the ear thermometer on different persons can be inappropriate in the event of certain acute infectious diseases because of the possible spread of germs despite cleaning and disinfection. If you have any doubts, please consult your doctor.
- This ear thermometer may only be used with the protective cap in place.
- The following circumstances may influence the temperature in the ear: if you have been lying on the ear for an extended period, the ears are covered, the ears are exposed to excessive heat/cold, or after swimming or bathing. In the event of one of these circumstances, wait 20 minutes before performing the measurement.
- As ear wax can affect the measurement, you should clean the ear before measuring if necessary.

1. Place a clean protective cap on to the sensor tip of the ear thermometer.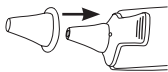


2. Press the ON/OFF button  to switch on the ear thermometer. Three acoustic signals sound and the display switches on.

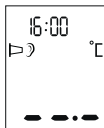
3. As the ear canal is slightly curved, you have to pull the ear slightly up and backwards before inserting the sensor tip. This is important so that the tip of the protective cap can be pointed directly at the eardrum.

4. Insert the sensor tip with the protective cap carefully into the ear canal and press the START button to begin the measurement.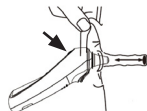

5. After approx. one second, a long acoustic signal sounds and the measurement is shown in the display. Press the ON/OFF button  to switch off the ear thermometer.

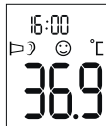Note

If no button is pressed for one minute after the measurement, the ear thermometer automatically switches itself off again.

Fever indicator

If the measured temperature is $< 38^{\circ}\text{C}$ (100.4°F), a laughing smiley 😄 appears on the display and a short acoustic signal sounds.


If the measured temperature falls between 37.5 and 37.9°C , two short acoustic signals will also sound (the indication for “raised temperature”).

If the measured temperature is $\geq 38^{\circ}\text{C}$ (100.4°F), a sad smiley ☹ appears on the display and a short, a long and three short acoustic signals sound one after the other.

7.2 Measuring object temperature


1. Remove the protective cap from the sensor tip of the ear thermometer (if it is on).

2. Press the ON/OFF button  to switch on the ear thermometer. Three acoustic signals sound and the display switches on.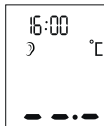

3. Ensure that you are in object temperature mode ().

To switch between body temperature and object temperature mode, briefly press and hold the ON/OFF button  and the START button at the same time.

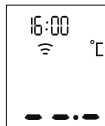
4. Hold the sensor tip of the ear thermometer 1 to 2 cm from the desired measuring point and press the START button.

5. After one second, a long acoustic signal sounds and the measurement is shown in the display. Briefly press the ON/OFF button  to switch off the ear thermometer.

7.3 Displaying room temperature


When you switch on the ear thermometer, the room temperature, the date and time appear alternately on the display every 3 seconds.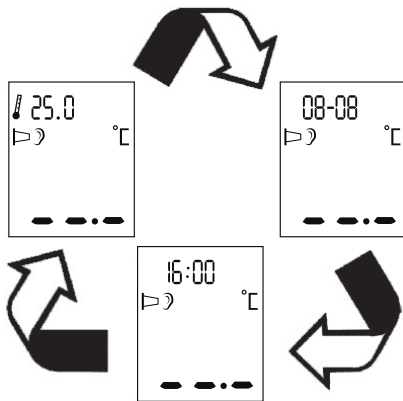

7.4 Displaying stored measurements

The ear thermometer automatically stores the last 10 temperature measurements of body and object temperature respectively (incl. date and time). To recall the stored measurements, proceed as follows:


1. Press the ON/OFF button  to switch on the ear thermometer.

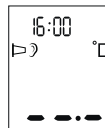2. Press and hold the ON/OFF button  for 3 seconds. An **M** and the last stored measurement appear on the display.

3. To switch between the individual stored measurements, press the ON/OFF button . With each subsequent pressing of the ON/OFF button , the memory space number is displayed first followed by the measurement.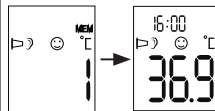

4. If you would like to proceed with a measurement, press the START button. Press and hold the ON/OFF button  for 3 seconds to switch off the ear thermometer.


8. Replacing the battery

Before replacing the battery, ensure that the device is switched off.


1. Replace the batteries as soon as the battery symbol () appears on the display.

2. Slide open the battery compartment and remove the used batteries from the battery compartment.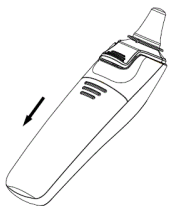

3. Insert two new 1.5 V AAA batteries. Make sure that the batteries are inserted the right way round. Close the battery compartment.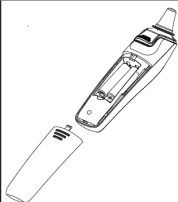

9. Cleaning and storage


Protective caps are only intended for single use.

The sensor tip is the most sensitive part of the ear thermometer. Work carefully when cleaning the sensor tip to avoid causing any damage. Use a clean cloth or cotton bud that can be moistened with disinfectant, alcohol or warm water.

To clean the entire device, use a soft cloth slightly moistened with a mild soapy solution.

Do not use any harsh cleaning products.

If prolonged storage is intended, you should remove the batteries. The device must not be stored or used at an excessively high or low temperature or humidity (see technical data), in sunlight, in association with an electrical current or in dusty locations. Otherwise inaccuracies can occur.

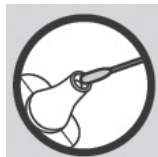10. Disposal

The empty, completely flat batteries should be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

Note: The codes below are printed on batteries containing harmful substances: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the unit at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

11. What if there are error messages?

Error message	Cause	Solution
ER1	The ear thermometer is not yet ready for measurement.	Take measurements only once body (🌀) or object temperature mode (🌡️) appears on the display.
ER2	The measured ambient temperature is not in the range from 10°C – 40°C (50 – 104°F).	Place the ear thermometer in a room with an ambient temperature between 10°C and 40°C (50°F and 104°F) for at least 30 minutes.
ER3	The ear thermometer is not placed in the ear correctly.	Please observe the notes in the chapter “Measuring”. Perform the measurement again.
ER4	The ear thermometer registered a large change in the ambient temperature.	Place the ear thermometer in a room with an ambient temperature between 10°C and 40°C (50°F and 104°F) for at least 30 minutes.

Error message	Cause	Solution
ERS	The ear thermometer is not working properly.	Replace the batteries and try again. If the fault occurs again, contact Customer Services.
ER6	You are in the “Measuring object temperature” mode and the protective cap is still on.	The protective cap must be removed to measure object temperatures.
H _i	In-ear measurement: the measured temperature is over 43°C (109.4°F). Object measurement: the measured temperature is over 100°C (212°F).	Ensure that the device and the protective cap are fully functioning. Perform the measurement again.
Lo	In-ear measurement: the measured temperature is below 34°C (93.2°F). Object measurement: the measured temperature is below -20°C (-4°F).	Ensure that the device and the protective cap are fully functioning. Perform the measurement again.
	The batteries are empty.	Remove the two empty batteries and insert two new 1.5 V AAA batteries into the ear thermometer.

If you cannot find a solution to your problem here, contact our Customer Services.

12. Technical data

Temperature range	• In ear: 34.0 – 43.0 °C (93.2 – 109.4 °F) • Object temperature: -20 – 100 °C (-4 – 212 °F) • Room temperature: 0 – 50 °C (32 – 122 °F)
Laboratory measurement accuracy	• In ear: ± 0.2 °C (0.4 °F) in measurement range of 35.5 – 42.0 °C (95.9 – 107.6 °F); ± 0.3 °C (0.5 °F) in measurement range of ≤ 35.5 °C (95.5 °F) and ≥ 42.0 °C (107.6 °F) • Object temperature: $\pm 4\%$ or ± 2 °C (4 °F) • Room temperature: ± 2 °C (± 4 °F)
Clinical reproducibility	0.11 °C (0.2 °F)
Duration of measurement	Approx. one second
Operating temperature range	10 – 40 °C (50 – 104 °F), humidity 15 – 85% (non-condensing)
Storage temperature range	-25 – 55 °C (-13 – 131 °F), humidity 15 – 95% (non-condensing)
Memory	10 memory spaces
Display resolution	0.1 °C or 0.1 °F
Battery	DC 3 V (2 x 1.5 V AAA batteries)
Dimensions	155 x 38 x 47 mm

Weight	Approx. 63 g (without batteries)
Protective caps	10
Automatic switch-off	60 seconds
Battery operating life	3,000 measurements in succession or one year with one to two measurements per day, including standby mode.
Safety classification	 Connected medical device of type BF
IP protection class	IP 22, protection against harmful ingress of water and dust

The serial number is located on the device or in the battery compartment.

13. Warranty/service

Hans Dinslage GmbH, Riedlinger Straße 28, 89524 Uttenweiler, Germany (hereinafter referred to as “HaDi”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

The warranty conditions below shall not affect the seller's statutory warranty obligations which ensue from the sales agreement with the buyer.

The warranty shall apply without prejudice to any mandatory statutory provisions on liability.

HaDi guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 2 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use.

German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, HaDi shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide HaDi, or an authorised HaDi partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable

batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);

- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by HaDi;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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Hans Dinslage GmbH, Riedlinger Str. 28, 88524 Uttenweiler, Germany